



Advanced Imaging Services

Computer & IT Specifications for iCatVision



Apple Mac will NOT operate the iCatVision software



Laptop Minimum Requirements

Intel Processor M 1.5Ghz

1GB of RAM NOTE 2GB will give peak performance

Windows XP Professional

40GB Hard Drive

CD/DVD Drive

Integrated Gigabit Ethernet 10/100/100 Network Connection



Workstation Minimum Requirements

Processor 2Ghz

1GB RAM NOTE 2GB will give peak performance

Windows XP Professional

Video Display capabilities of 1600x1200 resolution

18 or 19 inch Monitor recommended

40GB Hard Drive

CD/DVD Drive

Integrated Gigabit Ethernet 10/100/100 Network Connection

File Structure Setup

- 1 On the iCat Acquisition Computer C: drive create an iCatVision folder
- 2 Create a sub folder under iCatVision called iCatVision Watch
This is where the DICOM 3 data is exported. NOTE only DICOM files are placed here.
It is not recommended to have the Watch folder located on the server
- 3 On the server or central storage location, create a folder iCatVision